

Product # H84106601

Product # H84106601

Dale® Tracheostomy Tube Holder

#1
Just Got
Better

New Features:

- Fungus Resistant Barrier
- Tapered, Wider, More Secure Fastener Tabs
- Moisture Repellent Lining





Take Hold of Patient Care®

Dale® Tracheostomy Tube Holder

Now with Fungus Resistant Barrier




Product Description

Provides secure positioning of the tracheostomy tube. Tapered, wider fastener tabs are fungus resistant and are quickly and easily threaded through the flange of the tracheostomy tube. Two-piece neckband includes moisture repellent lining and a portion of stretch material to allow for edema or cough reflex.

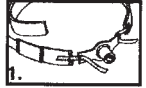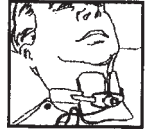

Application Instructions

1. Thread fastener tab through flanges on trach plate.
2. Bring tabs back over flange and adhere to the neckband, or back onto the soft material of the fastener tab.
3. Secure fastener tab of short section to the long neckband section. Adjust for proper length or compression.
4. Cut off excess material.

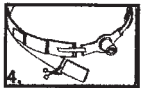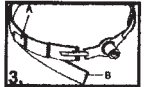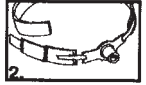

Caution: Use in supervised settings only. Monitor to prevent unintentional disengagement. **Do not wash.** For single patient use.

Ordering Information

H84106601 Dale® Tracheostomy Tube Holder
with Fungus Resistant Barrier

Fits most. Individually packaged in box of 10.

H84102401 Dale® Tracheostomy Tube Holder

Fits most. Individually packaged in box of 10.

H84102421 Dale® Tracheostomy Tube Holder
For Neonates/Infants

Fits neonate-infant.

Individually packaged in box of 10.

H84102431 Dale® Extension

6" extension tab for H84106601 or H84102401

HCPCS Code: A7526

Dale medical products are available from your preferred hospital/medical supply dealer. **Latex-free.**



Dale Medical Products, Inc.

7 Cross Street, PO Box 1556, Plainville, MA 02762

Tel: 800-343-3980 Int'l: 508-695-9316 Fax: 508-695-6587

www.dalemed.com

EU Representative:

MDSS GmbH, Burckhardtstr. 1

D-30163 Hannover, Germany

Tel: ++49-511-6262-8630 Fax: ++49-511-6262+8633