
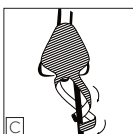
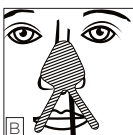
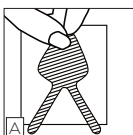


Dale®

NasoGastric Tube Holder

Application Instructions

1. After the NG tube is in place, make certain that the nose and the NG tube are both clean and dry. We recommend cleaning it per hospital protocol.
2. Peel back NG tube holder from liner starting at the top of the nose pad.(A)
3. Position and apply nosepad to the nose at a slight angle toward the nare that the NG tube has been inserted into.(B)
4. Choose the tab closest to the tube and begin to wrap it in a spiraling fashion so most of the NG tube is centered on the selected nare.(C)
5. Wrap the remaining tab using the same spiral technique.(C)
6. Change the tube holder after 3 days, or sooner if necessary.

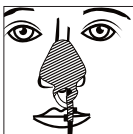

Removal

To remove the NG tube holder from the patient, simply pull back on the non-adherent tips on the end of each tab to unwrap them from the tube.

Continue to gently pull the tube holder in an upward fashion to remove from the patient's nose.

For single patient use. Disposable.

For an application video please visit www.dalemed.com

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Dale® is a registered trademark of Dale Medical Products, Inc. in the USA & EU.

TEL.: +1-508-695-9316
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

