


Adhesive Base

Application Instructions

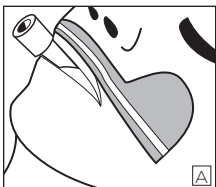

To prevent skin

ulcerations

or skin tears, assess

patient skin and medical

conditions prior to

applying. Monitor daily or more

frequently.

1. Apply skin prep.
2. Remove liner and place Adhesive Base at desired location **(A-D)**. Trim as needed.

Change base after 3 days, or sooner if necessary.

Removal

Adhesive should be removed with care to avoid skin trauma.

For an application video please visit www.dalemed.com

For single patient use. Disposable.

Not made with natural rubber latex.

Dale® is a registered trademark of Dale Medical Products, Inc. in the USA & EU.

TEL.: +1-508-695-9316
www.dalemed.com



MDSS GmbH

Schiffgraben 41

D-30175 Hannover, Germany

