
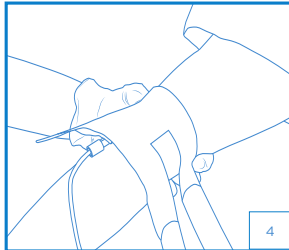
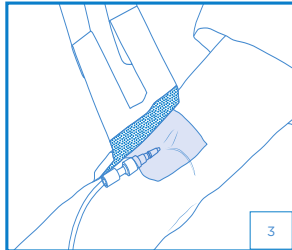



# Dale IV ARMOR®

Preserve the site, Guard against occlusion  
Flexible Protective Overlay


## Application Instructions

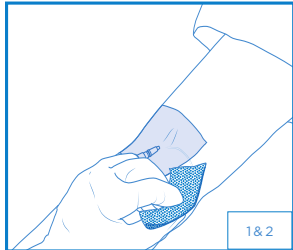


### Antecubital Application

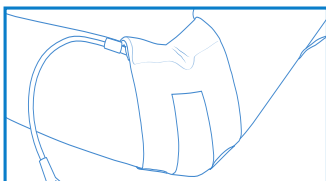

### Hand Application

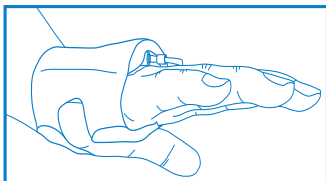


### Forearm Application

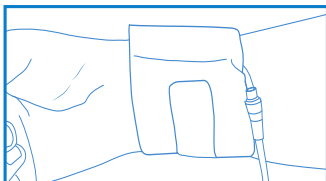

1. Secure and dress IV line per hospital protocol.
2. Apply skin prep evenly to areas adhesive will adhere to, and let dry for approximately 30 seconds.
3. Center the pad over the IV & dressing. Determine if length of legs need to be cut to prevent overlapping. If necessary, cut legs before removing liner.

\*Do not cut more than 2" from one side. If you require cutting more than 2", you must trim from other set of product legs.

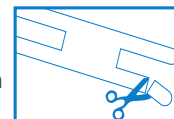

4. Pull back liner and press product to adhere to patient in one motion, one side at a time. Do not stretch to apply.



To inspect IV insertion site, lift center section of product to visualize area.

## Product Features

- Helps maintain a continuous IV flow while protecting and preserving the IV site
- Reduces the number of IV restarts, which decreases risk of infection
- Saves nursing time and reduces alarm fatigue
- Allows for patient movement while reducing downstream occlusion alarms caused by movement
- Improves patient comfort without the need for a stiff immobilizing device
- Reduces risk of dislodgement from picking and pulling of the IV by confused patients

### ORDERING INFORMATION

Product Number	Size	Quantity
675	One size fits most	10/Box

HCPCS Code A4221

Keep device clean and dry.  
Change after 3 days or sooner if necessary.

**Removal:** Device should be removed with care to avoid skin trauma. Re-apply skin prep if another device is to be applied.

**Dale**  
www.dalemed.com

Dale® is a registered trademark of Dale Medical Products, Inc. in the USA & EU.  
IV-ARMOR® is a registered trademark of Dale Medical Products, Inc. in the USA  
©2018 Dale Medical Products, Inc.

AD-045 REV C