

Dale[®] BreezeLock[™] Endotracheal Tube Holder

Application Instructions

- Clean tube. Let dry.
- Apply adhesive base. **(A)** Follow directions on 273 package.
- Apply device with track above upper lip and Tubing Channel over lips **(B)**. Press neckband onto hook portion of the base.
- Open the fastener tabs located at both ends of the neckband and wrap the band around the head just below the ears. Secure the fastener tabs to the outside of the neckband. **(C)**
- Position Tube Channel over the ET tube, twist key to the right to lock position along the track. **(D)**
 - wrap 6" strong adhesive tape 1x around the tube where the Channel will cover the tube
 - press Channel down over the tube
 - wrap tape around tube and Channel **(E)**

Alternate Taping Method:

- press Channel onto the tube
 - tightly wrap 6" strong adhesive tape around tube and Channel. **Wrap tape a minimum of 1" beyond the end of the Channel (E)**
- Ensure neckband security by readjusting the fastener tabs as necessary.

Tube Repositioning

Twist the key to the **left to unlock**. Slide tube to desired position. Twist Key to the **right to lock** the tube in place along the track.



Unlocked
Key is horizontal



Locked
Key is vertical

Change tape as necessary.

Change Adhesive Base after 3 days or sooner if necessary.

Removal

Device should be removed with care to avoid skin trauma. Re-apply skin prep if another device is to be applied.



To prevent skin ulcerations or skin tears, assess patient skin and medical conditions prior to applying. Monitor daily or more frequently.

Dale **recommends repositioning the endotracheal tube often** to help prevent injury to the lips and underlying tissues due to unrelieved pressure.

Dale **strongly recommends supporting the ventilator tubing** to reduce pressure on the patients face.

For single patient use. Disposable

Ordering Information

280	Dale[®] BreezeLock[™] Endotracheal Tube Holder Fits endotracheal tube sizes 7.0mm-10.0mm	10/Box
270	Dale[®] Stabilock Endotracheal Tube Holder Fits endotracheal tube sizes 7.0mm-10.0mm	10/Box
273	Dale[®] Adhesive Base	10/Box

Individually packaged.

Dale medical products are available from your preferred hospital/medical supply dealer.

US Patent #9,308,340 B2

Not made with natural rubber latex.

Dale[®] is a registered trademark of Dale Medical Products, Inc. in the USA & EU.



Dale[®]
www.dalemed.com

942197 REV A



Dale Medical Products, Inc.
40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298
T: 508-695-9316



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

