

# Dale®

Hold-n-Place®  
General Purpose Securement

## Application Instructions

1. After the line or tube is placed, clean & prep the skin per hospital protocol.
2. On some patients hair removal or skin prep may be required for better adhesion.
3. Make sure the skin is clean and dry before applying the securement device.
4. Lift the top fabric strap and remove **PULL** liner. **(A)**
5. Slide the securement device under the line or tube and center onto exposed adhesive. **(B)**
6. Place the top fabric strap over the line or tube and press onto the hook material. **(C)**
7. Pull one bottom release liner and press adhesive to skin, repeat for other side. **(D)**
8. Press securement device onto skin.
9. Replace securement device if soiled or saturated in fluids.

Tape-Strips are included for small & medium to help stabilize line or tube while applying the securement device.

## Removal

Use of an alcohol swab may be used to help with removal.

## Ordering Information

<b>950</b>	Small, Fits 1.5mm-4.5mm tubing	50/box
<b>951</b>	Medium, Fits 3mm-8mm tubing	50/box
<b>952</b>	Large, Fits 6mm-13mm tubing	50/box

**For single patient use. Disposable.**





For an application video please visit [www.dalemed.com](http://www.dalemed.com)

**STERILE R**

Dale medical products are available from your preferred hospital/medical supply dealer.

**Not made with natural rubber latex.  
Patent Pending**

Dale® & Hold-n-Place® are registered trademarks of Dale Medical Products, Inc. in the USA & EU.

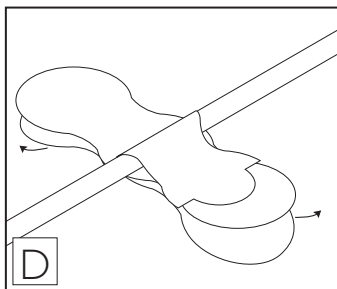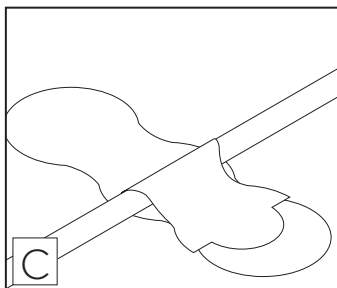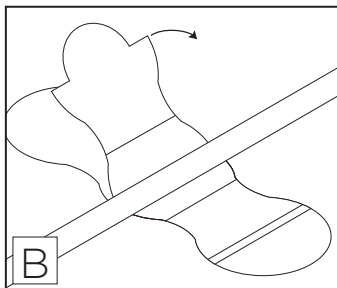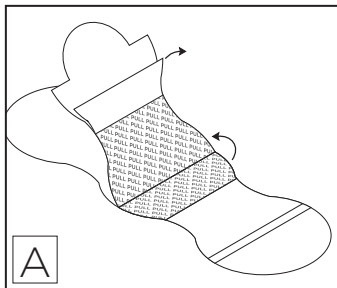

TEL.: +1-508-695-9316  
[www.dalemed.com](http://www.dalemed.com)