

PATIENT COMFORT AND
SATISFACTION SHOULD
NEVER BE COMPROMISED



think

of the ramifications

DALE® NASOGASTRIC TUBE HOLDER

RECOMMENDED APPLICATION INSTRUCTIONS

1. After the NG tube is in place, please make certain that the skin on the nose and the NG tube is both clean and dry. We recommend cleaning it per hospital protocol.
2. Peel back NG tube holder completely from liner starting at the top of the nose pad. Position and apply nosepad to the nose at a slight angle toward the nare that the NG tube has been inserted into. (see fig. 1)
3. Choose one of the two tabs closest to the nose and begin to tightly wrap the tube in a spiraling fashion around it so most of the NG tube is centered on the selected nare. (see fig. 2)
4. Wrap the second remaining tab using the same spiral technique. (see fig. 2)
5. Change the tube holder after 3 days, or sooner if necessary.

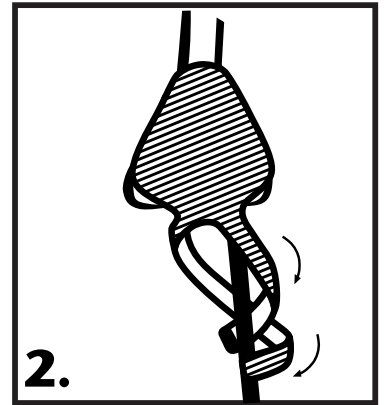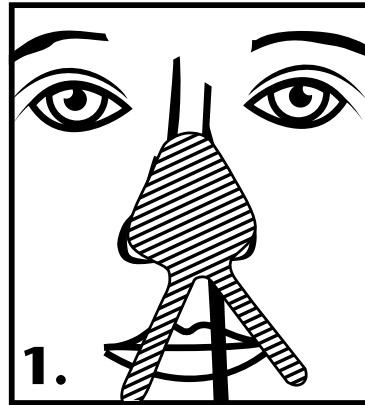

REMOVAL INSTRUCTIONS

1. To remove the NG tube holder from the patient, simply pull back on the non-adherent tips on the end of each tab to unwrap them from the tube.
2. Continue to gently pull the tube holder in an upward fashion to remove from the patient's nose.

***THIS IS A SINGLE PATIENT USE DISPOSABLE PRODUCT**

PRODUCT FEATURES

- Ideal for holding most nasogastric tubes, including nasoenteric, Salem Sump®, and Levin stomach tubes.
- Skin-friendly adhesive stays in place for up to 3 days, yet is easy to remove.
- Non-adherent plastic tips on both tabs make tube removal safer and easier when donning gloves.

SIZES AVAILABLE

Product Number	Size	Quantity
H84101601	Large / Adult	50 per box
H84101611	Small / Young Adult / Pediatric / Geriatric	50 per box



800-343-3980
www.dalemed.com

Dale® is a registered trademark of Dale Medical Products, Inc. in the USA & EU. Salem Sump® is a registered trademark of Covidien AG Corporation in Switzerland. ©2014 Dale Medical Products, Inc. AD-025 Rev B