

# DALE<sup>®</sup> ACE CONNECTOR<sup>®</sup> WITH ENFit<sup>™</sup> CONNECTION SYSTEM

infuse nutrients, medications, fluids, irrigate, check gastric residuals and control flow.

PRODUCT NUMBER

485

## AUTO-SEAL SYRINGE MED PORT

accepts most 60mL ENFit type tip syringes for irrigation, medication delivery and checking gastric residuals.\*\*

Align, Push & Twist ENFit 60mL syringe or Administration Set ENFit female connection into ENFit Auto-Seal Syringe Med Port with male ENFit Connection.

\*\*Medication must be crushed properly to prevent potential clogging.

## DISTAL END (PATIENT END)

closest to patient's stomach.

- universally connects to NG small bore tubes with ENFit connectors.

## ENFit FEMALE CONNECTOR CONNECTS TO:


- Small bore feeding tubes with male ENFit connectors.
- PEG, PEJ, and most G-J tubes with male ENFit connectors.
- Balloon replacement tubes with male ENFit connectors.

## ON/OFF HANDLE

controls fluid flow.

“ON” for feeding.

“OFF” to irrigate tube, check gastric residuals, administer medication, or ambulate patient.

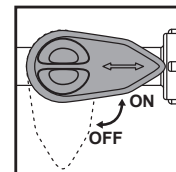

## PROXIMAL END (FEEDING BAG END)

furthest from patient.

## ENFit MALE CONNECTOR CONNECTS TO:

- enteral pump and gravity delivery sets with ENFit female connection.
- bolus feeding syringes with ENFit tip.

## THE ACE ADVANTAGE

- Supports a closed enteral system
- Protects clinician from potentially infectious splash-back
- Protects patient from outside contamination
- No soiled bed linens or loss of formula due to open adapters
- Provides auto-seal syringe med port seal for accepting medication, irrigation, or gastric residuals

Scan For More Info



MR CONDITIONAL

Note: the distal end is always on the side closest to the pt. This is true from a clinician's perspective. It is exactly opposite from a patient's point of view.

# Dale

800-343-3980

[www.dalemed.com](http://www.dalemed.com)

Dale<sup>®</sup> and ACE Connector<sup>®</sup> are registered trademarks of Dale Medical Products, Inc. in the USA & EU.

ENFit<sup>™</sup> is a registered trademark of GEDSA used with their permission.

©2017 Dale Medical Products, Inc.