

DALE[®] ACE CONNECTOR[®] WITH ENFit[®] CONNECTION SYSTEM

infuse nutrients, medications, fluids, irrigate, check gastric residuals and control flow.

PRODUCT NUMBER

485

AUTO-SEAL SYRINGE MED PORT

accepts most 60mL ENFit type tip syringes for irrigation, medication delivery and checking gastric residuals.**

Align, Push & Twist ENFit 60mL syringe or Administration Set ENFit female connection into ENFit Auto-Seal Syringe Med Port with male ENFit Connection.

**Medication must be crushed properly to prevent potential clogging.

DISTAL END (PATIENT END)

closest to patient's stomach.

- universally connects to NG small bore tubes with ENFit connectors.

ENFit FEMALE CONNECTOR CONNECTS TO:


- Small bore feeding tubes with male ENFit connectors.
- PEG, PEJ, and most G-J tubes with male ENFit connectors.
- Balloon replacement tubes with male ENFit connectors.

ON/OFF HANDLE

controls fluid flow.

"ON" for feeding.

"OFF" to irrigate tube, check gastric residuals, administer medication, or ambulate patient.

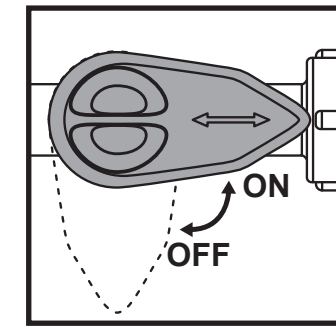

PROXIMAL END (FEEDING BAG END)

furthest from patient.


ENFit MALE CONNECTOR CONNECTS TO:

- enteral pump and gravity delivery sets with ENFit female connection.
- bolus feeding syringes with ENFit tip.

THE ACE ADVANTAGE

- Supports a closed enteral system
- Protects clinician from potentially infectious splash-back
- Protects patient from outside contamination
- No soiled bed linens or loss of formula due to open adapters
- Provides auto-seal syringe med port seal for accepting medication, irrigation, or gastric residuals

Scan For More Info

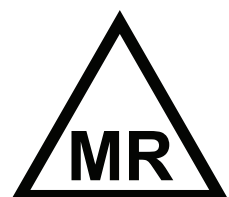

MR CONDITIONAL

Dale[®]

800-343-3980
www.dalemed.com

Dale[®] and ACE Connector[®] are registered trademarks of Dale Medical Products, Inc. in the USA & EU. ENFit[®] is a registered trademark of GEDSA used with their permission.
©2018 Dale Medical Products, Inc.

Note: the distal end is always on the side closest to the pt. This is true from a clinician's perspective. It is exactly opposite from a patient's point of view.