

THE DALE® ACE CONNECTOR® ENTERAL FAMILY



DALE OFFERS OPTIONS FOR EITHER
THE ENFit™ ISO 80369-3 STANDARD OR
LEGACY ENTERAL FEEDING SYSTEMS

Dale®
Dale Medical Products, Inc.

DALE® OFFERS ENTERAL FEEDING CONN

TRUST THE NEW DALE ACE CONNECTOR TO PROTECT YOU FROM SPLASH-

ORIGINAL DALE® ACE CONNECTOR® (FOR LEGACY ENTERAL APPLICATIONS)



THE ORIGINAL DALE ACE (ACCESS CONTROLLER FOR ENTERAL) CONNECTOR SUPPORTS MOST LEGACY ENTERAL SYSTEMS

- THIS CONNECTOR SUPPORTS CLOSED ENTERAL FEED SYSTEMS, AND CONTROLS FLUID FLOW INTO AND OUT OF LEGACY SYSTEMS, REDUCING THE RISK OF CROSS CONTAMINATION. THE DALE ACE CONNECTOR REPLACES 5 IN 1 CONNECTORS AND ENTERAL FEEDING Y-PORT ADAPTERS TO PROVIDE IN-LINE FEEDING, SUCTIONING, GASTRIC RESIDUALS, IRRIGATION, AND MEDICATION DELIVERY WITHOUT THE NEED TO DISCONNECT.
- THE DALE ACE CONNECTOR IS COMPATIBLE WITH LEGACY SYSTEMS, INCLUDING SALEM SUMP, PEG, J, MOST GJ 18 FRENCH AND LARGER, STANDARD FEEDING ADMINISTRATION SETS, SUCTION TUBING AND 60ML CATHETER-TIPPED ENTERAL SYRINGES.
- THE ACE ENTERAL CONNECTOR SYSTEM REMAINS CLOSED TO THE ATMOSPHERE FOR OPTIMAL INFECTION PROTECTION AS LONG AS IT IS CONNECTED TO A FEEDING CATHETER AND FEEDING ADMINISTRATION SET.
- THE ACE UTILIZES A UNIQUE SPLIT SEPTUM SYRINGE PORT SEAL THAT OPENS AND CLOSES UPON INSERTION AND REMOVAL OF MOST 60ML CATHETER TIP SYRINGES.

FACTORS TO MEET YOUR PRACTICE NEEDS

BACK WHILE PROTECTING YOUR PATIENT FROM EXTERNAL CONTAMINATION

THE DALE® ACE CONNECTOR® WITH ENFit™ TECHNOLOGY (FOR ISO 80369-3 ENFit ENTERAL APPLICATIONS)



THE NEW DALE ACE CONNECTOR SUPPORTS A CLOSED ENTERAL SYSTEM FULLY COMPATIBLE WITH THE ENFit ISO 80369-3 STANDARD.

- NEW AUTO SEAL SYRINGE MEDICATION PORT – UTILIZES NEW 80369-3 COMPATIBLE MED PORT TECHNOLOGY, WHILE INCORPORATING OUR SPLIT SEPTUM SYRINGE SEAL TECHNOLOGY FROM OUR TIME-TESTED ORIGINAL ACE CONNECTOR.
- NEW PURPLE COLOR CONSISTENT WITH OTHER ENTERAL INDUSTRY PRODUCTS.
- IMPROVED HANDLE DESIGN TO PREVENT MIS-INSERTIONS OF ENFit SYRINGE DURING LOW LIGHT ENVIRONMENTS.
- NEW ENFit COMPLIANT DISTAL (FEMALE PORT) AND PROXIMAL (MALE PORT) – WILL CONNECT TO ALL 80369-3 COMPLIANT ENTERAL TUBES INCLUDING PEG/PEJ, BALLOON REPLACEMENT PEG, MOST GJ TUBES, SMALL BORE FEEDING, AND FEEDING ADMINISTRATION SETS AND ENFit COMPLIANT SYRINGES.

THE DALE® ACE CONNECTOR® WITH ENFit™ CONNECTION SYSTEM

INFUSE NUTRIENTS, MEDICATIONS, FLUIDS, IRRIGATE, CHECK GASTRIC RESIDUALS AND CONTROL FLOW

AUTO-SEAL SYRINGE MED PORT

Accepts most 60mL ENFit type tip syringes for irrigation, medication** delivery and checking gastric residuals.


Align, Push & Twist ENFit 60mL syringe or Administration Set ENFit female connection into ENFit Auto-Seal Syringe Med Port with male ENFit Connection.

**Oral medications must be crushed properly to prevent potential clogging.

ON/OFF HANDLE Controls fluid flow

"ON" for feeding.

"OFF" to irrigate tube, check gastric residuals, administer medication, or ambulate patient.

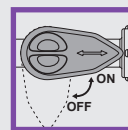

PROXIMAL END Furthest from patient

ENFit MALE CONNECTOR CONNECTS TO:

- Enteral pump and gravity delivery sets with ENFit female connection.
- Bolus feeding syringes with ENFit tip.

DISTAL END Closest to patient's stomach

- Universally connects to NG small bore tubes with male ENFit connectors.

ENFit FEMALE CONNECTOR CONNECTS TO:

- Small bore feeding tubes with male ENFit connectors.
- PEG/PEJ tubes with male ENfit connectors.*
- Balloon replacement G tubes with male ENFit connectors.
- Most G/J tubes with male ENFit connectors.

* Some PEG tubes may require use of a white ENFit transition adapter (supplied separately) to properly connect.

ORDERING INFORMATION

PRODUCT NUMBER	SIZE	QUANTITY
H84104851	ONE SIZE FITS ALL	50 PER BOX

OTHER RELATED PRODUCTS

DALE NASOGASTRIC TUBE HOLDER

H84101601	LARGE/ ADULT	50 PER BOX
H84101611	SMALL/YOUNG ADULT/PEDIATRIC/GERIATRIC	50 PER BOX

Dale DALE MEDICAL PRODUCTS, INC.
800-343-3980
WWW.DALEMED.COM